

DOCKNLOCK®

USER OPERATING INSTRUCTIONS



This manual must be passed on to the user/operator after installation.

Read instructions fully before use.

It is recommended that training be given on the proper use of this product.

Retain this manual for future reference.

Include it with the product in the event of transfer to new users.

Additional copies are available upon request.

Unwin House, The Horseshoe, Coat Road, Martock
Somerset, UK
Tel: +44 (0)1935 827740 Fax: +44 (0)1935 827760
Email: sales@unwin-safety.co.uk
www.unwin-safety.com

© 2009 C.N. UNWIN LTD. ALL RIGHTS RESERVED

CONTENTS

- A. Features
- B. Docking your wheelchair
- C. Releasing your wheelchair
- D. Frequently Asked Questions
- E. Maintenance
- F. Guarantee
- G. Safety
- H. Warrantee Registration Form

A. FEATURES

DOCK N LOCK[®] is simple to operate and provides an effortless way of securing your wheelchair into your vehicle.



B. DOCKING YOUR WHEELCHAIR

1. To dock your wheelchair, roughly align it to the driving position and move forward over the docking station until you feel the unit engage.
2. To confirm that the wheelchair is fully locked into the docking station, turn the vehicle ignition switch so the ignition lights come on but the engine is not started. If no audible alarm sounds and the green light on the release button is illuminated, the wheelchair is fully docked.



NOTE: YOUR WHEELCHAIR IS ONLY SECURED IF THE RELEASE BUTTON IS GREEN.

3. If an audible warning sound is heard and the release button flashes red, your wheelchair is not fully engaged into the docking system. In this case reverse the wheelchair and re-enter the docking station. Ensure that no there is no alarm sound and the release button is now green.



4. When the wheelchair is fully docked, turn the wheelchair power off and apply the brakes.
5. Fit an appropriate passenger restraint.

NOTE: As a failsafe, the audible alarm will sound for seven seconds when the vehicle power is switched on, even if there is no wheelchair engaged. The alarm will then turn off for the entire time the vehicle ignition is switched on.

C. RELEASING YOUR WHEELCHAIR

1. Turn the vehicle ignition off.
2. Unfasten the passenger restraint.
3. Turn the wheelchair power on and release the brakes.
4. Press the release button and reverse the wheelchair away from the docking station.



D. FAQ's

Should I advise my insurance company?

Yes, your insurance company should be advised of any modifications made to the vehicle. There will normally be no extra premium required.

Does the DOCK N LOCK[®] require servicing?

No, the DOCK N LOCK[®] does not require servicing. Keep both unit and the area around it clean and free from loose items.

E. MAINTENANCE

The DOCK N LOCK[®] is designed to be maintenance free, however please ensure that the area in front of the docking station is kept clear from any loose items that could be entangled.

F. WARRANTY

The DOCK N LOCK[®] system is guaranteed against defect in workmanship and materials for a period of three years. In the event of any defect being discovered, please contact in the first instance the company who fitted the unit.

This guarantee is limited to the repair or replacement of the faulty unit, unless agreed in advance with Unwin Safety Systems.

For full terms and conditions, please contact Unwin Safety Systems.

G. SAFETY

- The Information in this manual is provided with your convenience and safety in mind. Please read and retain these user instructions to ensure a trouble free and safe operation.
- Always travel in the wheelchair manufacturer's recommended travelling position. Do not make any adjustments to the position of the wheelchair whilst the vehicle is in motion. For further information, please refer to the wheelchair manufacturer's recommendations.
- Ensure you understand the operating characteristics of the DOCK N LOCK[®] system, especially the warning light sequence.
- **DOCK N LOCK[®] is intended as a wheelchair restraint only. A suitable occupant restraint should be used when travelling in a wheelchair.**