

## **HOME AND AWAY HARNESS**

HAHJ, JUNIOR VERSION  
HAHI, INFANT VERSION

### **USER OPERATING INSTRUCTIONS**



**Read instructions fully before use.**

**It is recommended that training be given on the proper use of this product.**

**Retain this manual for future reference.  
Include it with the product in the event of transfer to new users.  
Additional copies are available upon request.**

Unwin House • The Horseshoe • Coat Road  
Martock • Somerset • TA12 6EY • ENGLAND

Tel: +44 (0)1935 827740 • Fax: +44 (0)1935 827760

Email: [sales@unwin-safety.co.uk](mailto:sales@unwin-safety.co.uk)

[www.unwin-safety.com](http://www.unwin-safety.com)

© 2011 C.N. Unwin Ltd. All rights reserved

UI1532B



# CONTENTS

- A. Limitation of use
- B. Fitting attachments to the vehicle seat
- C. Fitting Occupant Harness
- D. Removing the Harness
- E. Maintenance

## A. Limitation of use

The home and away harness **must always be used in combination with the standard seatbelt fitted to the vehicle seat.**

### VERSIONS AVAILABLE

“**Junior** “ and “**Infant**” versions are available, their specifications are:

#### Junior

From shoulder to waist, 508mm fully extended.  
Waist 1321mm fully extended.  
Strap anchorage attached to seat 1727mm long.

#### Infant

From shoulder to waist, 356mm fully extended.  
Waist 737mm fully extended.  
Strap anchorage attached to seat 1727mm long.




## B. Fitting attachments to the vehicle seat

Secure the anchorage straps to the vehicle seat as illustrated, ensure the buckles are in the correct positions, the strap widths are not too wide for the occupant, and the vertical straps are tensioned.



## C. Fitting the Occupant Harness

1. Place the harness over the shoulders of the passenger with the yellow band to the front at waist level.
2. Fasten the two Velcro straps together at their back securely.

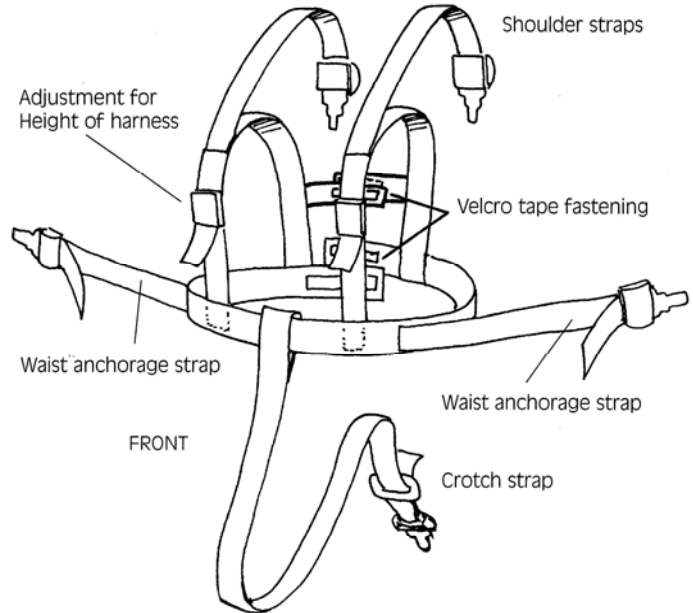

3. Position the passenger onto the seat ensuring they are sitting comfortably. (A child booster seat should be used where required).



4. Attach the two top straps to the buckles on the top of the seat facing forward.



5. Attach the two waist straps to the lower side buckles on the seat.
6. Adjust the fit on each side by pulling the loose webbing through their webbing adjusters adjacent to the buckles.
7. Ensure the harness is unable to ride up the seat back.



## C. Fitting the Occupant Harness

8. Fit the crotch strap end into the middle buckle on the rear of the seat.
9. Tension the upper belts on the torso with their individual positions, and the crotch strap.

**When correct, fit the standard vehicle seatbelt.**

## D. Removing the Harness

1. Reverse instructions.
2. Note the tension is released on the two adjusters on the occupant upper body belts by holding between the fingers and thumb and pressing together.



## E. MAINTENANCE

1. Regularly inspect the straps for damage, wear or malfunction. If any problems are identified, harness must be replaced immediately.
2. All webbing and components can be cleaned as necessary, care should be taken to prevent contamination of the webbing with polish, oils and chemicals.
3. To clean the straps use warm soapy water and a clean soft cloth. Rinse with clear water, and allow to air dry.
4. To disinfect use a mild spray disinfectant - do not use products containing bleach. Important: when cleaning or disinfecting do not immerse buckles or rail fittings in disinfectant or water.
5. If the vehicle is involved in an accident when any restraints are deployed remove them from service and replace immediately.



Unwin House • The Horseshoe • Coat Road  
Martock • Somerset • TA12 6EY • ENGLAND  
Tel: +44 (0)1935 827740 • Fax: +44 (0)1935 827760  
Email: [sales@unwin-safety.co.uk](mailto:sales@unwin-safety.co.uk)  
[www.unwin-safety.com](http://www.unwin-safety.com)